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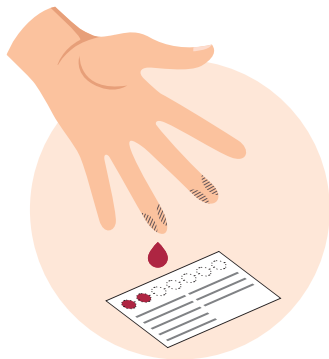
ANLEITUNG

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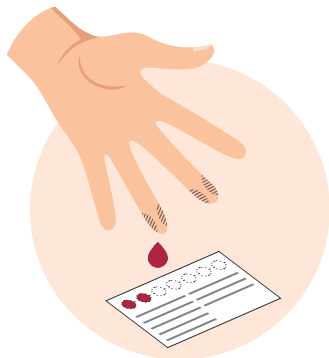
GUIDELINE

1a Blood collection from the fingertip


- Wash hands thoroughly with soap, disinfect middle or ring finger with alcohol or isopropanol, if necessary. Do not use any disinfectant.
- If required, massage or warm up the hand or fingers, use commercially available lancing devices (safety lancets with a wide cutting surface).
- Puncture the fingertip at the side and wipe the first drop of blood with a clean swab.
- Apply blood on pre-printed filter paper circles, without touching the card with the finger.
- Fill 1-7 circles completely, depending on the analysis requested.

Caution: Apply a large drop of blood onto each circle to ensure the circles are filled as completely as possible. The blood must soak through the card so that blood spots look almost equal on both sides. Apply blood onto the card from one direction only. Avoid multiple blood applications (overloading).

1b Alternatively: Venous blood collection

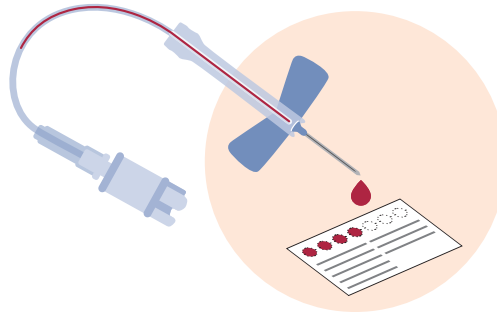Only qualified medical professionals may carry out venous blood collection. Hygienic and medical instructions must be strictly adhered to. Apply blood directly from the collection system onto the dried blood card or use a pipette to transfer it from the collection tube. Do not use any additives (EDTA, heparin, etc.).

Front

blood applied correctly

card packed/
dispatched wet

too little blood
applied

blood did not soak
through the card

blood applied too thickly
(clotted)

Back

2 Drying

Allow to dry at room temperature for at least 2-4 hours by placing the card upright/inclined against an object. Do not dry the card with a radiator, a hairdryer or with exposure to sunlight!

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Blood spots:
Best quality essential!
Fill at least 5 circles completely with one single very large drop of blood each from this side only.
No EDTA!

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3 Preparation for shipment

- Fill out data fields of request form and dried blood card completely.
- Put the request form and/or laboratory referral letter and dried blood card in an envelope, stamp sufficiently and send as soon as possible.
- Avoid mailboxes exposed to strong sunlight or cold, consider date and time of postal collection. Sending from a post office or centre is recommended.

Screening-Labor Hannover

P.O. Box 91 10 09
30430 Hanover

Fon: 05108-92163-0
Fax: 05108-92163-19

Copyright 2020 – All contents, especially text, pictures and graphics are governed by the copyright laws of Germany. Any duplication, processing, distribution, translation or any form of utilisation beyond the scope of copyright law shall require the prior written consent of the Screening-Labor Hannover. For questions or additional information, please contact the laboratory.

Design: NF.DESIGN / Nina Fischer